

UROSENSOR™ INSTRUCTIONS

TESTING: Every night before use test the Urosensor™ and the DRI Sleeper® alarm.

Plug the Urosensor™ into the DRI Sleeper® alarm and slowly insert the Urosensor™ into a container of **hot water** (this roughly approximates urine, although urine is much more conductive), and see how many pairs of the black sensing bars need to be inserted before triggering the DRI Sleeper®. If more than three pairs of bars need to be inserted, then clean the Urosensor™ by brushing well with soap and water using a soft brush (ie: a used toothbrush).

Test again and it should now trigger after immersion of one or two pairs of the sensing bars. If the washing did not improve the functioning, then it is time to purchase a replacement Urosensor™.

The Urosensor™ is a consumable item and so it is strongly recommended that a spare Urosensor™ is purchased so that there is no interruption in treatment if the Urosensor™ needs to be replaced.

For hygiene reasons the Urosensor™ is **not returnable**.