



Uro-Bond® IV

Brush-on Silicone Adhesive



DESCRIPTION

Uro-Bond® IV brush-on silicone adhesive is a pressure-sensitive silicone adhesive specifically formulated for unique bonding applications. Uro-Bond® IV is suitable for adhering non-permanent material to the body surface. Uro-Bond® IV's features are: non-hazmat (no special handling required), non-flammable, virtually odor-free, retention of adhesive qualities in the presence of moisture or perspiration, adhesive properties are unaffected by normal temperature variations or time and it adheres to many surfaces—skin, metals, glass, paper, fabric, plastics, silicone and organic rubber products.

APPLICATIONS

Uro-Bond® IV brush-on silicone adhesive may be used to adhere dressings, prosthetics and metal or plastic devices to the body. Suggested uses include but are not limited to: Securing male external urinary catheters, adhering colostomy, ileostomy and tracheostomy applications, attaching electrodes to the skin for patient monitoring, affixing surgical dressings and pads to the skin and adhering maxillo-facial prosthetic devices. Products made in part or otherwise incorporating Uro-Bond® IV brush-on silicone adhesive should be thoroughly tested before use to determine the acceptability of the product's performance in a specific application for his/her own use.

STORAGE AND SHELF LIFE: Do not use or store near fire, flame, heat, electric sparks or any other source of ignition.

Use only as directed. Keep out of eyes. In case of eye contact, immediately flush eyes with water, remove any contact lenses and continue to flush eyes with plenty of water for at least 15 minutes. Contact physician. Harmful or fatal if ingested. Do not inhale or permit adhesive to enter stoma or open wounds. In case of accidental ingestion, give fluids liberally and consult with local POISON CONTROL CENTER. If redness, swelling or rash appears near area of use, discontinue use. Contact Physician. **Keep out of reach of children.** When stored in its original unopened container at or below 72° F (22° C), Uro-Bond® IV has a shelf life of Two (2) years from the date of manufacture. If product thickens during storage, Urocare recommends the use of Uro-Bond® IV Thinner (reorder No. 5014 03) to return Uro-Bond® IV to the desired consistency. PLEASE NOTE: Do not leave jar open for prolonged periods of time—close cap securely immediately after application.

INSTRUCTIONS FOR USE

When used as supplied, Uro-Bond® IV is applied using the brush attached to the underside of the cap. To obtain the maximum adhesion capability of Uro-Bond® IV, surfaces must be thoroughly cleaned and allowed to dry thoroughly prior to application. The adhesive should be applied liberally to both surfaces which are to be adhered; additional coats of adhesive may increase adhesion.

REMOVAL

To remove Uro-Bond® IV from the body surface, use Urocare Adhesive Remover Pads (Reorder No. 5600) or another medical tape and adhesive remover. Do not use Uro-Bond® Thinner as an adhesive remover; it is strictly intended for use as a thinning agent only! To remove Uro-Bond® IV from surfaces other than the body (skin), use one of the following removers or solvents: isopropylalcohol (rubbing alcohol), ethyl alcohol, toluene, xylene, or hexane; however, it should be noted that these removers or solvents may cause skin irritation and are highly flammable. They may also deteriorate certain surfaces like certain types of plastics, and should be used accordingly. It is the user's responsibility to thoroughly test the use of any remover or solvent to determine suitability of use. Liberally apply the appropriate remover to a cloth or gauze pad and wipe the surface clean.

PACKAGING

Uro-Bond® IV brush-on silicone adhesive is supplied in a 3 fl. oz. (88.7 ml.) and 1 ½ fl. oz. (44.4 ml.) glass jars with a brush attached to the underside of the cap and is secured with a tamper proof seal.

ORDERING INFORMATION

For current information about Urocare Products, please refer to our web site: www.urocare.com. For the location of your nearest Urocare distributor, email us at sales@urocare.com or call: (909) 621-6013 weekdays from 8:00 to 4:30 P.M. (PST).

The information and data contained herein are based on information we believe to be accurate and reliable; however, Urocare encourages you to thoroughly test any application and independently conclude the effectiveness and suitability of Uro-Bond® IV before use. Suggestions for use should not be taken as inducements to infringe upon any patent.

