



UNIVERSAL FABRIC LEG STRAPS

For use with most vinyl, latex and rubber urinary leg bags

- Provides a superior fit over standard latex leg straps
- Fully adjustable self-gripping fasteners make application safe and simple
- Elastic sections make them flexible and comfortable
- Durable, reusable and hand or machine washable

6390
One Pair
Fits 8" thru 24" (20 cm thru 61 cm)

† Approximate dimensions in inches(") and centimeters(cm)

IMPORTANT! READ THIS FIRST:

For best results, take certain precautions before using these straps. Be certain that there is no loose stitching or other irregularities before each use. Those with known allergic reactions to certain materials (especially neoprene rubber) should not wear these straps. Discontinue use at first signs of dermatitis or any other type of skin irritation and contact a physician immediately.

CAUTION: Do not over-tighten straps; products made with elastic and/or neoprene rubber material can—if improperly fit—cause circulation restriction or skin breakdown; however, a bag strapped too loosely can slip down the leg and cause damage to male external or Foley catheters by stretching them. Should any discomfort occur while wearing, discontinue use immediately and contact a fitter or physician. To avoid possible drainage problems, read all instructions and precautions carefully before use. Only allow a urinary drainage bag to fill to its maximum capacity in emergency situations.

APPLICATION INSTRUCTIONS:

1. Be certain the bottom drain valve/thumb clamp is closed to prevent spills.
2. Wrap the straps around the leg on either the thigh or calf. If the straps are too long, trim by cutting only the elastic section between the loop to achieve the desired length.
3. Thread the ends of the straps with the Urocare® logo through the leg bag eyelets so that the straps run behind the leg bag as shown in Figure 1.
4. Place the leg bag in the desired location on the leg and fasten the ends of the straps together—the straps should be snug yet comfortable.

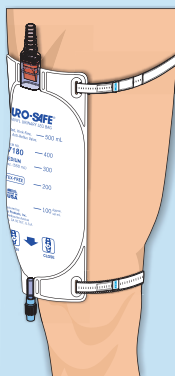


FIGURE 1

WASHING INSTRUCTIONS:

Close straps (Velcro® fasteners) before washing. Hand or machine wash cold—gentle/delicate. Do not bleach. Do not tumble dry. Do not wring. Dry in shade. Do not iron. Do not dry clean.



For more information on this or any other Urocare product, please visit our website at www.urocare.com. To place an order, please contact your health care provider.



Urocare Products, Inc.
2735 Melbourne Avenue
Pomona, CA 91767-1931
U.S.A.



US and Canada: +1 (800) 423-4441
Outside the US: +1 (909) 621-6013



sales@urocare.com

LEARN MORE



MADE IN
CHINA



PLEASE
RECYCLE

FN: P16390EN-69601101D